

Alaris® Syringe module-compatible syringes and flow rate ranges

Alaris® products

Medications delivered on Alaris Syringe module must be prepared in a syringe that accommodates the desired flow rate. The following chart shows the flow rate ranges for each syringe size. In addition, the compatible syringe sizes with reorder numbers are displayed.

Syringe size	Brand and model numbers				
	BD	Monoject	Terumo	IVAC®	Flow rate range
1 mL	309628	N/A	N/A	N/A	0.01 to 30 mL/hr
3 mL	309657	SoftPack only: 1180300777	SS-03L	N/A	0.01 to 100 mL/hr
5 mL	309646	N/A	SS-05L (6 mL total)	N/A	0.1 to 150 mL/hr
6 mL	N/A	SoftPack: 1180600777 Rigid Pack: 8881516937	SS-05L (6 mL total)	N/A	0.1 to 150 mL/hr
10 mL	309604* 300912	N/A	SS-10L (12 mL total)	N/A	0.1 to 250 mL/hr
12 mL	N/A	SoftPack: 1181200777 Rigid Pack: 8881512878	SS-10L (12 mL total)	N/A	0.1 to 250 mL/hr
20 mL	302830	SoftPack: 1182000777 Rigid Pack: 8881520657	SS-20L SS-20L2	N/A	0.1 to 500 mL/hr
30 mL	302832	N/A	SS-30L (35 mL total)	N/A	0.1 to 650 mL/hr
35 mL	N/A	SoftPack: 1183500777 Rigid Pack: 8881535762	SS-30L (35 mL total)	N/A	0.1 to 650 mL/hr
50 mL	N/A	N/A	N/A	30602N	0.1 to 999 mL/hr
60 mL	309653	SoftPack: 1186000777 Rigid Pack: 8881560125	SS-60L	N/A	0.1 to 999 mL/hr

*Syringes manufactured only in the United States

Alaris System is manufactured by CareFusion Corporation USA; refer to the Infusion Pump Directions For Use

© 2013 CareFusion Corporation or one of its subsidiaries. All rights reserved. Alaris, IVAC, CareFusion and the CareFusion logo are trademarks or registered trademarks of CareFusion Corporation or one of its subsidiaries. All other trademarks are property of their respective owners. IF1302 (0113/1500) AL303/Feb14/v1

CareFusion Australia
Ph: 1800 333 372

CareFusion New Zealand
Ph: 0508 422 734

carefusion.com

