


## Adhesive Base

### Application Instructions

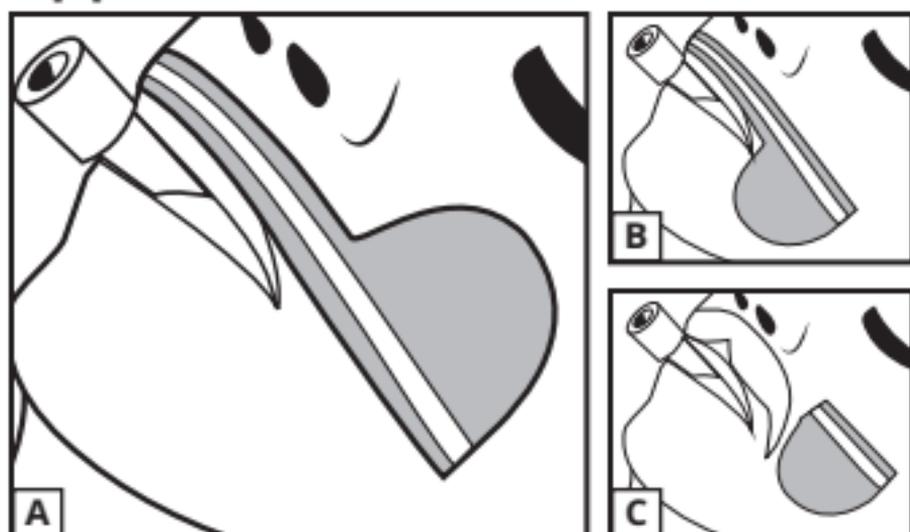

1. Apply skin prep.
2. Remove liner and place Adhesive Base at desired location (**A-C**). Trim as needed.

Change base after 3 days, or sooner if necessary.

Monitor daily or more frequently.

### Removal

Adhesive should be removed with care to avoid skin trauma.

To prevent skin ulcerations or skin tears, assess patient skin and medical conditions prior to applying.

For single patient use. Disposable

For an application video please visit [www.dalemed.com](http://www.dalemed.com)

Dale medical products are available from your preferred hospital/medical supply dealer.

Not made with natural rubber latex.

Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK



**Dale Medical Products, Inc.**  
40 Kenwood Circle • Suite 7  
Franklin, MA 02038-3298, USA  
T: +1-508-695-9316  
[www.dalemed.com](http://www.dalemed.com)



**MDSS GmbH**  
Schiffgraben 41  
D-30175 Hannover, Germany



**MDSS CH GmbH**  
Laurenzenvorstadt 61  
5000 Aarau, Switzerland

