

DALE BREEZELOCK®

TAPING INSTRUCTIONS

For best tube security with any type of cloth tape it is best to wrap tape around the tube 1x BEFORE pressing the Tubing Channel down onto the tube.

As per the Instructions for Use:

To secure tube into Tubing Channel



Wrap strong adhesive tape 1x around the tube where the Tubing Channel will cover the tube.



Press Tubing Channel down over the tube.



Wrap tape around tube and Tubing Channel.



Wrap tape a minimum 1" beyond the end of the Tubing Channel.



View Application
Video

Dale
Medical Products



508-695-9316 • dalemed.com

Dale & BreezeLock are registered trademarks of Dale Medical Products, Inc.
©2023 Dale Medical Products, Inc. All rights reserved.

DALE BREEZELOCK®

REPOSITIONING INSTRUCTIONS

Dale recommends repositioning the endotracheal tube often to help prevent injury to the lips and underlying tissues due to unrelieved pressure.

As per the Instructions for Use:

To reposition the tube along the track



Tube Repositioning While holding the taped CHANNEL with one hand, lift DALE OVERLAY off TRACK.



Slide CHANNEL to move tube to desired location.



Press DALE OVERLAY back down onto TRACK and neckband to secure the tube's position in place.



Monitor daily or more frequently. Reposition tube per facility protocol.



View Application
Video

Dale
Medical Products



508-695-9316 • dalemed.com

Dale & BreezeLock are registered trademarks of Dale Medical Products, Inc.
©2023 Dale Medical Products, Inc. All rights reserved.