

User height  
600-1800 mm



Max. load  
114 kg



Environment  
Indoor & outdoor

# Cruiser

The Convoid Cruiser is a 30° fixed-tilt pediatric wheelchair that is convenient, lightweight and compact folding. The Cruiser is custom-built to meet each individual requirements. It comes equipped with Convoid's proprietary self-tensioning seating system, Self-Tensio™, which provides increased seating tolerance and can help promote neutral pelvic positioning.

## Highlights about Cruiser



### Customisable growth capabilities

Adjustable seat depth extends chair life for a growing child.



### 30° fixed tilt

Increased tilt angle improves postural control through upper body positioning.



### Foldable

The Cruiser is folded quickly and fits into almost any car.



### Self-Tensio™

Built in self-tensioning seating system increases sitting tolerance.



## Standard, Cruiser Standard

Solid and foldable frame, fixed tilt, anatomic back, flip-up armrest, 2-point positioning belt, off road tires, castor lock, toggle wheel locks, attendant brake and swing-away foot plates.


## Standard, Cruiser Scout

Solid and foldable frame, 30° fixed tilt, seat depth growth, one-piece angle adjustable push brace (CX12-CX16) or two-piece push handle (CX18) 2-point positioning belt, quick release rear wheels, castor lock, toggle wheel locks and removable adjustable swing-away foot plates.

## Options

Size:	Model	Tyres
- CX10	- Standard with foot brake	- Solid
- CX12	- Scout	- Air
- CX14	- Standard with attendant hand brake	- Solid w/transp. fitting
- CX16		- Air w/transp. fitting
- CX18		

### Cushion colours

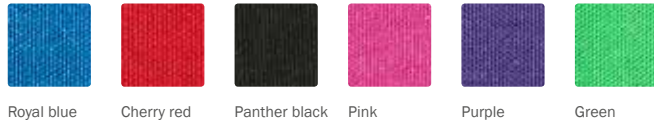

### Frame colour














Cruiser Scout 16"



Cruiser standard 18"

## Measurements

		<b>CX 10</b>	<b>CX 12</b>	<b>CX 14</b>	<b>CX 16</b>	<b>CX 18</b>
	Effective seat width	255 mm	305 mm	355 mm	405 mm	455 mm
	Effective seat depth	152-299 mm	200-330 mm	254-380 mm	279-406 mm	356-533 mm
	Backrest height	530 mm	560 mm	640 mm	710 mm	640 mm
	Total mass	12 kg	12 kg	13 kg	14kg	14 kg
	Max. load regular/transit	34/30 kg	34/30 kg	45,5/45,5 kg	77/77 kg	114/77 kg
	Overall width	560 mm	600 mm	650 mm	700 mm	724 mm
	Length	1220 mm	1120 mm	1245 mm	1320 mm	1400 mm
	Overall height	940 mm	970 mm	940 mm	1020 mm	970 mm
	Folded length	1190 mm	1220 mm	1190 mm	1350 mm	1270 mm
	Folded width	380 mm	380 mm	410 mm	410 mm	380 mm
	Folded height	430 mm	430 mm	430 mm	430 mm	430 mm

Further measurements at our website

### Nice to know

- The Cruiser is lightweight and has an easy three dimensional fold, which mean it requires minimal storage space and is easy to take along.
- The individual angle adjustable footplates allows for optimum leg and foot positioning.






# Accessories



**Extended headrest cover**  
 With or without windows (Canopy)  
 CX10 C904736/C904720  
 CX12 C904737/C904721  
 CX14 C904738/C904722  
 CX16 C904739/C904723  
 CX18 C904740/C904724

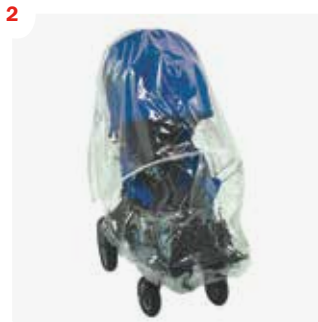

**Rain cover**  
 CX10-14 C904939  
 CX16-18 C904910



**Headrest extension**  
 Curved cordura (black)  
 CX10 C903332  
 CX12 C902863  
 CX14 C900484  
 CX16 C900138  
 CX18 C902619



**Headwings**  
 Padded:  
 CX10 C903331  
 CX12 C902862  
 CX14 C900681  
 CX16 C900142  
 CX18 C902618  
 Occi: C904779



**3-pt positioning belt**  
 C904017  
 With depth adjustment croch strap




**H-harness with padded covers**  
 C903988  
 3-pt position belt required



**Full support torso vest**  
 CX10 C903330  
 CX12 C902861  
 CX14 C900680  
 CX16 C900339  
 CX18 C902616



**Soft lateral support**  
 Single/Double flap  
 CX10 C903328/C903329  
 CX12 C902859/C902860  
 CX14 C900678/C900679  
 CX16 C900140/C900204  
 CX18 C902614/C902615

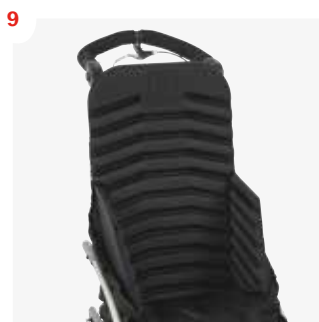

**Reducer seat insert**  
 Seat width 5 cm smaller, seat depth 2,5 cm smaller  
 CX10 C904211  
 CX12 C904111  
 CX14-18 C904110



**Calf panel**  
 C904580



**Medial thigh support-abductor**  
 with position/align cushion  
 CX10 C903326  
 CX12 C902857  
 CX14 C900480  
 CX16 C900137  
 CX18 C902607



**Lateral thigh support-abductor**  
 with position/align cushion  
 CX10 C903327  
 CX12 C902858  
 CX14 C900482  
 CX16 C900188  
 CX18 C902608



**Angle adjustable footplates**  
 CX10 C903445  
 CX12-14 C903268  
 CX16-18 C903269



**Footplate securement strap**  
 C903584



**Foot positioners**  
 Small (18-25 cm) C903415  
 Medium (20-28 cm) C903416  
 Large (23-30 cm) C903417




**Upper extremity support surface**  
 Clear tray  
 CX10 C903078  
 CX12 C903385  
 CX14 C903386  
 CX16 C903387  
 CX18 C903388



**Anchor transit kit**  
 CX10-16 C997125  
 CX18 C997141  
 Scout model:  
 CX12-16 C997219  
 CX18 C997220



**Anti-tip**  
 CX10-12 C904520  
 CX14-16 C904540

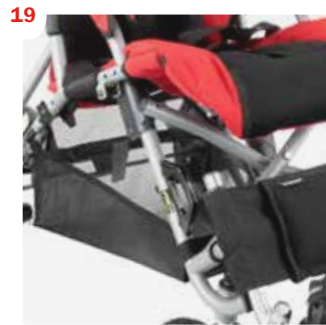


**Under seat storage basket**  
 CX10-12 C904520  
 CX14-16 C904540  
 CX18 C904048




**Utility bag**  
 C904531



**Saddle bag**  
 C904094

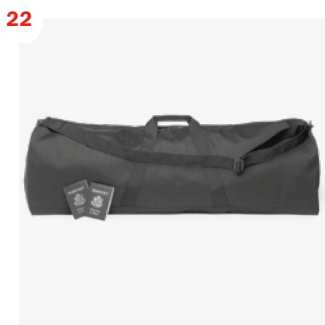

**Wheelchair travel bag**  
 C902022

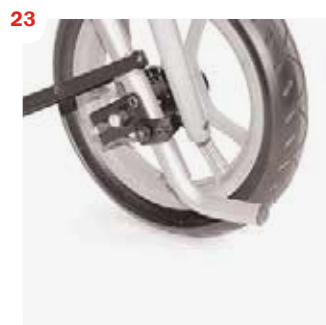

**Curb tipper**  
 CX10-16 C904128  
 CX18 C903916  
 Scout: C903916



**Incontinence cushion liner**  
 C902324