

Cuticell[®] Contact

Silicone Wound Contact Layer



- Atraumatic and gentle
- Single sided adherence
- Highly flexible

Cuticell® Contact

Cuticell Contact is a gentle and highly conformable silicone wound contact layer for pain-free and atraumatic dressing changes. It's perforated transparent structure enables easy and effective wound management.

Atraumatic and gentle

Once applied, Cuticell Contact can remain in place for up to 14 days, protecting the wound. Exudate can easily pass through the perforations with minimal risk of tissue ingrowth. As the adhesive is only on the wound contact side, secondary dressings can be easily applied and changed without causing trauma to the wound bed.

- Pain-free, atraumatic dressing changes¹
- Undisturbed healing
- Comfortable to wear




Exceptional performance

Crystal clear

Cuticell Contact's transparency aids wound management.

No need to remove dressing to examine the wound.

- Can stay in place while secondary dressings are changed

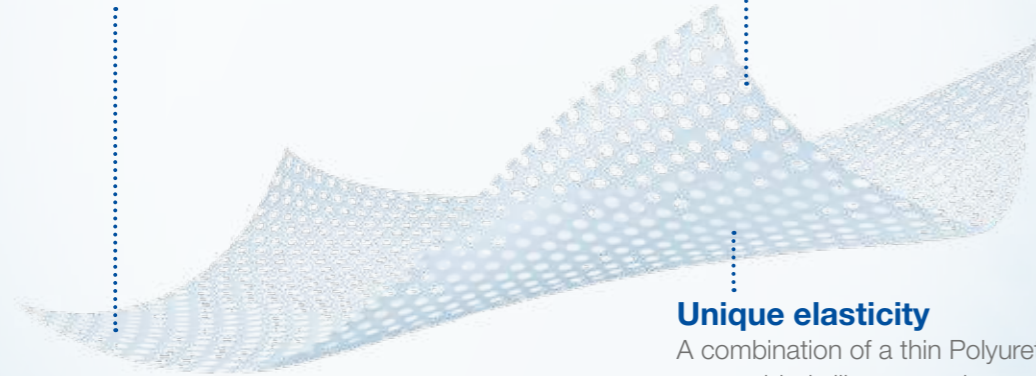

Optimal perforated structure

With up to 10 perforations per cm².

Ensures even highly viscous exudate can easily pass through.

- Helps prevent skin maceration^{1,2}
- For wounds with low to high exudate levels

Unique elasticity

A combination of a thin Polyurethane-film and a one-sided silicone coating.

Highly flexible and conformable.

- Comfortable to wear
- No slippage
- Easy application

Easy application



Step 1

Remove the overlapping protection film from the adhering face of the dressing.



Step 2

With the adhesive face down, apply Cuticell Contact, making sure to overlap wound margin by 2cm.



Step 3

Now remove the second protective film from the dressing surface and smooth gently with your hand.

NB: If several dressings are needed, make sure that they DO NOT overlap; obstructed pores can prevent exudate from passing through which may result in maceration of the peri-wound skin.

Siltec technology

Cuticell Contact benefits from Siltec technology. The silicone coated wound contact side only adheres to dry healthy skin, therefore offers very gentle adhesion.

- Optimal protection of skin and wound
- Secure adhesion without sticking to wound bed¹
- Atraumatic and pain-free removal
- Gentle removal without residue^{2,3}



Step 4

Can be used with a wide range of absorbent secondary dressings depending on the patient's needs, e.g. super-absorbent dressing Cutisorb® Ultra.

A selection of adhesive and non-adhesive fixation is available from BSN medical.



Cuticell[®] Contact

Indications

Cuticell Contact is suitable for use on the following wound types:

- Cuts
- Lacerations
- Abrasions
- Blisters
- Burns
- Skin lesions
- Chronic wounds
(in conjunction with secondary absorbent dressings)
- Also appropriate for skin protection
(e.g. after radiation treatment)
- Suitable for especially sensitive and fragile skin

Ordering information

Size	Pack Qty	Code		
		BSN	NHS	PIP
5cm x 7.5cm	1 x 5	72680-00	ELA677	388-6447
7.5cm x 10cm	1 x 5	72680-01	ELA678	388-6454
10cm x 18cm	1 x 5	72680-02	ELA679	388-6462
15cm x 25cm	1 x 5	72680-03	ELA680	388-6470



References:

1. DOF Preventing hypergranulation, protecting epithelialisation
2. DOF Skin Test No. 2
3. DOF Skin Test No. 1



BSN medical Ltd. • PO Box 258 • Willerby • Hull • HU10 6WT
Tel: 01482 670192 • Fax: 01482 670190

E-mail: advancedwoundcare.uk@bsnmedical.com • www.bsnmedical.co.uk

© Registered trade mark © BSN medical limited 2014 0515/714