

The **Dale Stabilock Endotracheal Tube Holder** secures endotracheal tubes quickly, easily and with maximum stability.

**Dale**<sup>®</sup>  
Always Better



## Stabilock Endotracheal Tube Holder

**HELPS PREVENT ACCIDENTAL EXTUBATION, ALLOWS ACCESS TO MOUTH FOR ORAL CARE AND OFFERS MANY OTHER CLINICAL AND PATIENT ADVANTAGES.**

The Dale Stabilock Endotracheal Tube Holder provides reliable stabilization of endotracheal tubes (sizes 7.0-10.0mm) following intubation. It helps prevent accidental extubation especially during patient transport.

### ADHESIVE BASE

With four placement options for its Adhesive Base, the Dale Stabilock Endotracheal Tube Holder covers the full spectrum of patient needs -comfortably and securely.



- Easy tube repositioning
- Gentle on delicate skin with no irritating-to-remove residue
- Can be left in place for up to 3 days
- Easily modified and simple to apply
- Not made with natural rubber latex.

### TUBE CHANNEL

- Allows easy access for oral care
- Prevents kinking or flattening of tube
- Minimizes ET tube movement and helps prevent accidental extubation



### RECOMMENDATIONS

- Reposition the endotracheal tube often to help prevent injury to the lips and underlying tissues due to unrelieved pressure.
- Support the ventilator tubing to reduce pressure on the patient's face.
- To prevent skin ulcerations or skin tears, assess patient skin and medical conditions prior to applying. Monitor daily or more frequently.

#### ORDERING INFORMATION

Product Number	Size	Quantity
270	Stabilock Fits endotracheal tube sizes 7.0mm-10.0mm	10 per box
273	Adhesive Base	10 per box

US Patent #8,096,300

**Dale**  
www.dalemed.com

Dale® is a registered trademark of Dale Medical Products, Inc. in the USA & EU.  
©2017 Dale Medical Products, Inc.

PS-270 REV G