

JOBST[®]

Bellisse

Bellisse[®] is exclusively distributed by JOBST[®]

BELLISSE[®]

COMFORTABLE, SUPPORTIVE
TREATMENT FOR BREAST OEDEMA



THERAPIES. HAND IN HAND.

BSN medical

an SCA company

BELLISSE

INNOVATIVE APPROACH TO BREAST AND CHEST WALL OEDEMA FOLLOWING BREAST CANCER TREATMENT

The welcomed integration of Bellisse now means that BSN medical can provide effective treatment solutions to help patients live their lives as independently as possible.

Bellisse was developed by Lesli R. Bell, DPT, CLT-LANA, a lymphedema nurse specialist. Her interest in women's health and oncology led her to pursue an idea for a compression garment for the management of lymphedema of the chest wall. Bellisse* is the first compression product made especially for the needs of the post breast cancer patient with secondary breast lymphoedema or post-surgery breast oedema.



FITTED

TO YOUR THERAPY
TO YOUR LIFESTYLE
TO YOUR EVERY DAY AND NIGHT

FOR LIFE

UNIQUE FEATURES AND BENEFITS

BECAUSE COMFORT MAKES A DIFFERENCE IN A WOMAN'S EVERYDAY LIFE

The Bellisse is an innovative treatment approach for chest wall, breast oedema and/or fibrosis following breast cancer or surgery. Don't confuse this revolutionary "compression garment" with a mastectomy or post-operative bra. This is a medical compression garment that helps to reduce discomfort and provides gentle, even compression around the torso, facilitating drainage of excess lymphatic fluid from the breast and chest wall area. It is available in a large range of sizes and available in two colors, buff and black.

The Bellisse can be worn day and night, is fully adjustable, and provides gentle compression with or without the addition of JOBST® JoViPads. A number of anatomically shaped pads are available and can be used in combination with the Bellisse.



1 The natural stretch cups have an inner pocket to accommodate the lateral pad or the **Chest-Wall Pocket Pad**.

- Compression promotes lymph drainage



2 Special **Compressure Comfort®** inner support system provides circumferential compression.

- Comfort, day and night



3 Secure front zipper with hooks and eyes to hold garment in place during zipping.

- For easy donning and doffing



4 Soft, non-roll lower band keeps garment from riding or rolling up.

- Soft to the skin
- Smooth while sleeping
- Unobtrusive under clothing



5 Highly adjustable with three position back closure and with extender providing additional 3 rows of eye loops.

- Maximum adjustability



6 Padded straps are adjustable.

- Comfortable wearing without shoulder pressure

Bellisse is made of Polyester, Nylon and Spandex

* Bellisse® and Compressure Comfort® are registered trademarks of Lightning2, LLC.


HOW TO MEASURE FOR A BELLISSE

GETTING THE RIGHT FIT

It is recommended that measurements are taken when the oedema is at its most reduced state, often this is found first thing in the morning.

Measurement Points

1. Measure in cm the circumference at Point ❶. Located directly under the bust (where a regular bra sits)
2. Move 5 - 7cm down towards the rib cage from Point ❶ and measure circumference at Point ❷
3. To determine the band size, take the larger measurement of Point ❶ & Point ❷
4. Measure the circumference at Point ❸ at the fullest part of the bust
5. To determine the cup size, subtract the bust measurement from the band size. Use the difference in the chart below to select the cup size

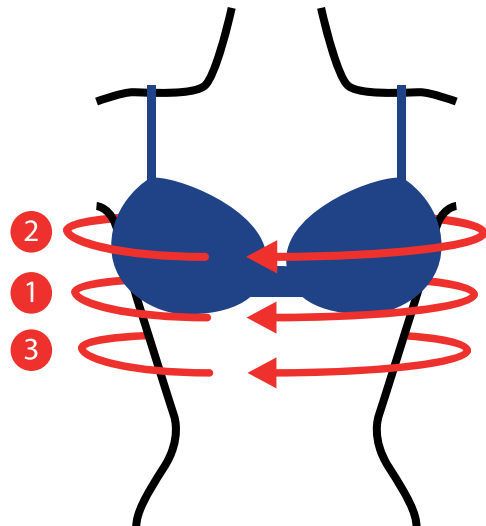

Cup Size	A/B	C/D	DD/E
	5cm or less	5 - 10cm	10 - 15cm

If more than 15cm, go up one one band size

Band measurement (cm)	75 - 80	81 - 85	86 - 90	91 - 95
Bellisse Band Size	30	32	34	36
Band measurement (cm)	96 - 100	101 - 105	106 - 110	111 - 115
Bellisse Band Size	38	40	42	44
Band measurement (cm)	116 - 120	121 - 125	126 - 130	
Bellisse Band Size	46	48	50	

ORDERING INFORMATION

THE RIGHT SIZE FOR YOUR PATIENT

Bellisse sizing chart will aid you to determine the best fit providing comfortable support and effective compression.

Bellisse Size	Black	Buff
30 A/B	N/A	76125-29
30 C/D	N/A	76125-30
32 A/B	76125-00	76125-31
32 C/D	76125-01	76125-32
32 DD/E	76125-02	76125-33
34 A/B	76125-03	76125-34
34 C/D	76125-04	76125-35
34 DD/E	76125-05	76125-36
36 A/B	76125-06	76125-37
36 C/D	76125-07	76125-38
36 DD/E	76125-08	76125-39
38 A/B	76125-09	76125-40
38 C/D	76125-10	76125-41
38 DD/E	76125-11	76125-42
40 A/B	76125-12	76125-43
40 C/D	76125-13	76125-44
40 DD/E	76125-14	76125-45
42 A/B	76125-15	76125-46
42 C/D	76125-16	76125-47
42 DD/E	76125-17	76125-48
44 A/B	76125-18	76125-49
44 C/D	76125-19	76125-50
44 DD/E	76125-20	76125-51
46 A/B	76125-21	76125-52
46 C/D	76125-22	76125-53
46 DD/E	76125-23	76125-54
48 A/B	76125-24	76125-55
48 C/D	76125-25	76125-56
48 DD/E	76125-26	76125-57
50 C/D	76125-27	76125-58
50 DD/E	76125-28	76125-59

JOBST JOVIPADS

WORKING IN COMBINATION

Available in different shapes the JOBST JoViPads address particular areas of the breast and chest wall affected by pitting edema and/or fibrosis. The pads provide gentle and prolonged pressure to soften congested tissue and encourage evacuation of lymph fluid to functional pathways. Designed for use under the Bellisse.



Lateral Pad



Mini Axilla Pad



Chest-Wall Pocket Pad



Post-Lumpectomy Pad



Serratus Anterior Pad



Unilateral Post-Mastectomy Pad



Double Mastectomy Pad



Axilla Pad



Inframammary Pad



Cleavage Pad



Drain Pocket Pad

JOBST JoViPads are available in both Black and Buff variants and a number of sizes ranging from small to extra large.

ORDERING INFORMATION

Cleavage Pad		
	Black	Buff
One Size	75928-00	75928-01
Lateral Pad		
	Black	Buff
Small	76166-00	76166-03
Medium	76166-01	76166-04
Large	76166-02	76166-05
Axilla Pad		
	Black	Buff
30-38cm - S/M	75925-00	75925-02
38-46cm - L/XL	75925-01	75925-03
Mini-Axilla Pad		
	Black	Buff
Small	75926-00	75926-03
Medium	75926-01	75926-04
Large	75926-02	75926-05
Extra Large	75926-07	75926-06
Inframammary Pad		
	Black	Buff
Small	75927-00	75927-04
Medium	75927-01	75927-05
Large	75927-02	75927-06
Extra Large	75927-03	75927-07
Serratus Anterior Pad		
	Black	Buff
Small	75929-00	75929-04
Medium	75929-01	75929-05
Large	75929-02	75929-06
Extra Large	75929-03	75929-07

Post Lumpectomy Pad		
Cup Size	Black	Buff
A/B - Small	75931-00	75931-04
C - Medium	75931-01	75931-05
D/DD - Large	75931-02	75931-06
E - Extra Large	75931-03	75931-07
Unilateral Post Mastectomy Pad		
	Black	Buff
Small	75932-00	75932-04
Medium	75932-01	75932-05
Large	75932-02	75932-06
Extra Large	75932-03	75932-07
Double Mastectomy Pad		
	Black	Buff
Small	75933-00	75933-03
Medium	75933-01	75933-04
Large	N/A	75933-05
Chest-Wall Pocket Pad		
	Black	Buff
Small	75934-00	75934-04
Medium	75934-01	75934-05
Large	75934-02	75934-06
Extra Large	75934-03	75934-07
Drain Pocket		
	Black	Buff
One Size	76126-00	76126-01
Bellisse Extender		
	Black	Buff
One Size	76127-00	76127-01



INTEGRATED THERAPY SOLUTIONS FROM BSN MEDICAL

BSN medical connects its broad portfolio in lymphology, non-invasive orthopaedics and wound / vascular management with strong technological competencies to meet the market's need for reduced complexity and more effective management. With a continuously developing, unique and holistic **network of integrated therapy solutions**, BSN medical consistently supports a **continuum of care**.

WOUND / VASCULAR

LYMPHOEDEMA

CHRONIC VENOUS DISEASE

PRIMARY WOUND CARE

ORTHOPAEDICS

ANKLE SPRAIN

DISTAL RADIUS FRACTURE

BSN medical Limited

PO Box 258 • Willerby
Hull • HU10 6WT

www.bsnmedical.co.uk
Tel: 01482 670100 • Fax: 01482 670111
E-mail: orders.uk@bsnmedical.com

????/???

BSNmedical

an SCA company