



# Kangaroo™ Joey Enteral Feed and Flush Pump


## ALARMS / ERROR CODE INFORMATION

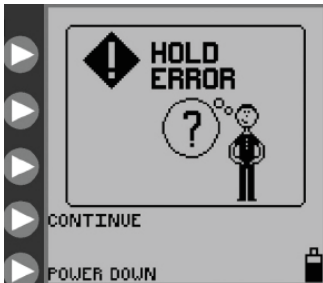

**HOLD ERROR:**  
Pump left in Hold for more than 10 minutes. Press **CONTINUE** followed by **RUN** to start pump running or power down pump.

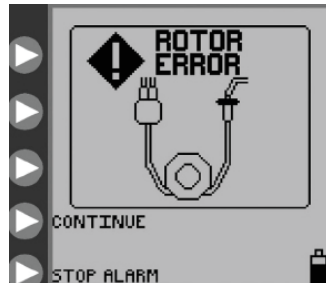

**ROTOR ERROR:**  
Problem with pump set tubing around the rotor. Ensure silicone tubing is correctly loaded around rotor.

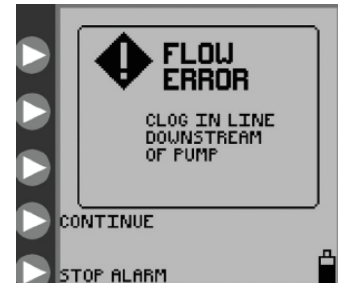

**FLOW ERROR:**  
Obstruction in line downstream from pump. Check for kinked or blocked tubing below pump and rectify.




**FEED ERROR:**  
Feed container empty or obstruction in the line upstream from pump. Check for kinked tubing or obstruction in line above pump or attach new/refilled feed container.



**FLUSH ERROR:**  
Water container empty or obstruction in the line upstream from pump. Check for kinked tubing or obstruction in flush line above pump or refill flush container.




**FEEDING COMPLETE:**  
Programmed feed has been delivered. Power down pump.

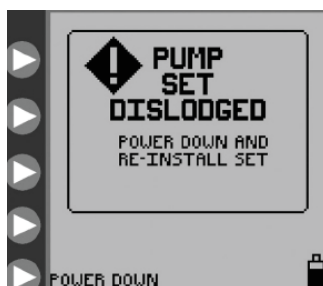

**PUMP SET DISLODGED:**  
Black retainer ring (MISTIC) is not properly loaded in MISTIC pocket. Ensure pump set is correctly loaded.



**BATTERY LOW:**  
Battery must be recharged immediately. Plug power cord into AC outlet.

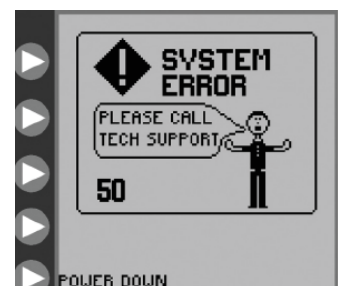

**SYSTEM ERROR:**  
General error – see pump manual. Send pump for service.