

# Kangaroo™ Joey Enteral Feed and Flush Pump

## USER GUIDE *(Backpack)*



**1**

Load set into pump as per 'Pump Set Loading Procedure' in training manual.



**2**

Place pump into elasticated pouch on bottom right of pack.



**3**

Pass both ends of feeding set through opening on left side of pouch.



**4**

Attach feeding set to feed container.



**5**

Attach feed container to clips at top of backpack.



**6**

Pass end of feeding set which will connect to the feeding tube through the opening on the left-hand side. Ensure medication port is closed.



**7**

Connect the feeding set to the feeding tube and commence feeding regime as per clinician's instructions.

Open front pouch of backpack to view settings if required.