



Step-by-step
application guide

AirLife® noninvasive ventilation full face mask

Follow these instructions to properly secure the AirLife® noninvasive ventilation (NIV) full face mask.

1. Remove the mask and headgear from the packaging.
2. Disconnect one or both of the headgear buckles from the mask.
3. Place the headgear over the patient's head and adjust the upper headgear straps to a comfortable tension. (A)
4. Hook the plastic buckle(s) onto the mask frame. (B)
5. Adjust the upper headgear straps to a comfortable tension. (C)
6. Adjust the lower headgear straps to a comfortable tension. (D)
7. Ensure the overhead strap is in the proper position to avoid slippage. (E)
8. Adjust the forehead pad by squeezing the fins at the top of the mask. Proper adjustment aids in mask stability and minimises pressure points. (F)
9. Connect the circuit and begin NIV therapy.
10. Check for excessive leaks and adjust the fit of the mask as necessary.

To take the mask off the patient, disconnect one or both buckles from the mask and remove.

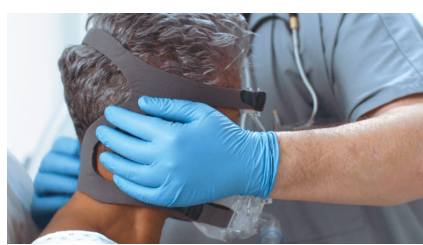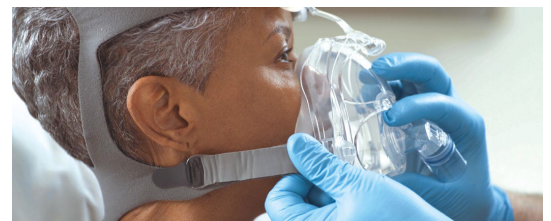

The flexible swivel elbow of the AirLife NIV mask reduces the torque applied to the seal between the mask and patient, helping to minimise pressure points.



Mask not flexed



Mask flexed