

Step 1



- Lift the line and SecurAcath® for 360 degree cleaning to be achieved on every dressing change.
- Do not twist or rotate the SecurAcath®.
- Lifting the SecurAcath® regularly may aid easy removal.

Step 2



- Clean 360 degrees around the line and SecurAcath® with wipes or applicators as per Trust policy.

Step 3



- Pay particular attention to the insertion site.
- The line will remain stable during cleaning.

Step 4




- Clean the top and back of the SecurAcath®, the surrounding skin and line.
- Allow to air dry completely.
- Loosely apply a clear film dressing.

Hints & Tips



- Use sterile forceps to remove debris between the line and the SecurAcath®.
- Dried blood can be lifted by applying saline soaked gauze for a few minutes
- If patient complains of acute pain from the SecurAcath®, the device may not be sited correctly and will need reviewing.
- SecurAcath® is bio-patch compatible.

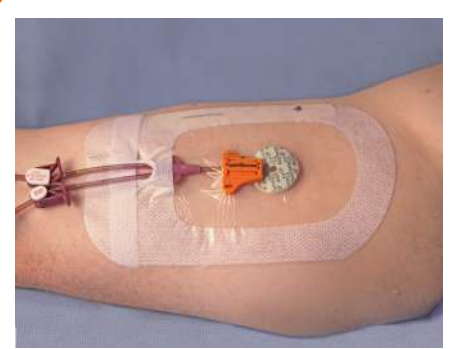

Distributed By Aquilant

Aquilant House, Unit B1-B2 Bond Close,
Kingsland Business Park, Basingstoke,
Hampshire, RG24 8PZ, United Kingdom
T: +44 (0) 1256 306 506
E: contactus@aquilantservices.com

Interrad Medical

181 Cheshire Lane,
Suite 100, Plymouth,
MN 55441 USA
+1.763.225.6699
www.securacath.com



SecurAcath® app!

