

- If insertion site is dry or firm, apply saline soaked gauze to the site for 3-5minutes, this will soften the skin aiding easier removal.
- Catheter can be withdrawn and haemostasis achieved before or after securAcath is removed (as per local protocol)

Step 1



- Grip the HOLD tab with thumb and finger of one hand to stabilise device and securement feet beneath the skin
- Pry upward at the edge of LIFT tab with index finger tip of other hand and thumb placed at center to release cover from base

Step 2



- Completely detach cover from base


FOLD OPTION Can be used with or without catheter in place

Step 3



- Gently pull skin above SecurAcath taut
- Fold wings downward to bring feet together under the skin

Step 4

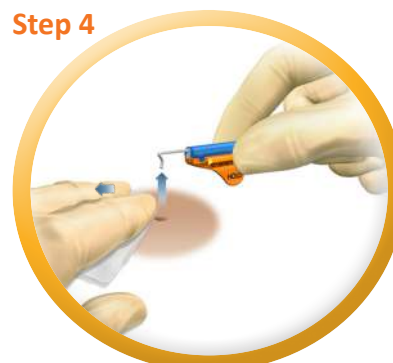

- Hold folded base horizontal to skin
- Use swift, deliberate upward motion to remove

SPLIT OPTION Catheter must be removed prior to using this method

Step 3



- Use a blunt tip scissors to cut base completely in half lengthwise along the groove

Step 4



- Gently pull skin above SecurAcath taut
- The flexible securement feet are shaped like an "L" with the feet extending 5mm to each side of the insertion site

Step 5



- Turn blue edge upward and use a swift, deliberate upward motion to remove each foot separately