



## Application Guide

### Midlines and Central Lines



***Prior to application of SecurePortIV™, ensure all skin prep solutions are dry and bleeding has stopped.***

- Point the tip of the applicator towards the ceiling and away from the patient
- Press the bottom of the applicator upward to break the seal




- Invert applicator and gently squeeze the ridges to initiate adhesive flow
- Apply 2-3 very small drops at the catheter skin junction, allowing it to run around the catheter
- Wait 7 seconds between each drop
- Ensure the adhesive surrounds the entire circumference of the catheter



- If the catheter hub is close to the insertion site, you may place 1-2 drops under the catheter hub and wings
- If sutures are present, the adhesive can be applied over the suture site



- Adhesive will be completely dry within 3-5 minutes
- Apply transparent film dressing as per hospital protocol

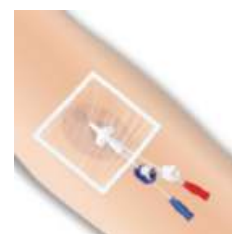

#### Tips

To prevent oozing post procedure, ensure bleeding has stopped completely prior to product application. Less is More! The product dries within 1 minute but only when a very thin layer is applied

#### Removal

Removal of adhesive occurs naturally over 5 – 7 days. Earlier removal may be accomplished by gently pressing adhesive remover pad over the insertion site for 30-60 seconds. This will soften the adhesive and allow easy removal of the device.



**HC21**  
Healthcare

For more information please visit: [www.adhezion.com](http://www.adhezion.com)  
or contact us at [ukorders@hc21.eu](mailto:ukorders@hc21.eu)

For additional information & support please contact your local Territory Manager



Jennifer Miller  
National Clinical Specialist  
E: [jennifer.miller@hc21.eu](mailto:jennifer.miller@hc21.eu)  
M: +44 7796 270 937



Matthew Ryder  
South London & Sout East  
E: [matthew.ryder@hc21.eu](mailto:matthew.ryder@hc21.eu)  
M: +44 7825 014 291



Daniella Lake  
Central  
E: [daniella.lake@hc21.eu](mailto:daniella.lake@hc21.eu)  
M: +44 7880 827 208



Bridie Power  
London & Essex and South  
E: [bridie.power@hc21.eu](mailto:bridie.power@hc21.eu)  
M: +44 7557 571 587



Scott Clissold  
Wales & South West  
E: [scott.clissold@hc21.eu](mailto:scott.clissold@hc21.eu)  
M: +44 7557 302 742



Philippa Betts Dawson  
Scotland & North East  
E: [philippa.bettsdawson@hc21.eu](mailto:philippa.bettsdawson@hc21.eu)  
M: +44 7552 234 809

**Distributed by: HC21**  
Unit B1-B2 Bond Close  
Kingsland Business Park,  
Basingstoke, Hampshire, RG24 8PZ

T: 01256 365 450  
E: [ukorders@hc21.eu](mailto:ukorders@hc21.eu)  
[www.healthcare21.eu](http://www.healthcare21.eu)