



Medi-Vac™ Flex Advantage™ Single-Canister

Set-up process

Set-up

-  Before use, visually inspect the liner for perforation or other damage. Also, inspect the reusable outer canister for wear or other damage. Discard product if damage is observed.
-  Do not use if product is past its "Use by" date.



1 Place Cardinal Health™ Medi-Vac™ Reusable Outer Canister into bracket on roll stand or wall mount to be used in vertical orientation only.



2 Remove liner from the packaging, then fully extend and twist the flexible liner.



3 Insert the liner into the same size Cardinal Health™ Reusable Outer Canister.



4 Press down firmly on the lid to ensure proper seal and listen for a "click".



5 Connect the red, braided tube to the VACUUM Port using a "push and twist" motion



6 Attach the patient tubing to the PATIENT Port.



7 Verify that all of the connections are secure and caps are pressed firmly in place.



8 Turn suction ON using on/off valve on the Medi-Vac™ Flex Advantage™ Bracket or the regulator valve.

Medi-Vac™ Flex Advantage™ Single-Canister

Shutdown process



1 With suction still ON disconnect the patient tubing from the PATIENT Port and securely cap the PATIENT Port.



2 With suction still ON disconnect the red, braided tube from the VACUUM Port and securely cap the VACUUM Port.



3 Ensure vacuum remains ON until all ports are securely capped. Turn suction OFF using on/off valve on the Medi-Vac™ Flex Advantage™ Bracket or the regulator valve.



4 Remove liner by lifting upward on the rim of the lid. Liners are disposable and should not be reused. Dispose of in compliance with hospital protocol.



5 Remove liner by lifting upward on the rim of the lid. Liners are disposable and should not be reused. Dispose of in compliance with hospital protocol.



6 Do not discard the reusable outer canister. Reusable components should be cleaned and disinfected in accordance with facility procedures.

Shutdown

For healthcare professionals only.

Important information: Prior to use, refer to the instruction for use supplied with this device for indications, contraindications, side effects, suggested procedure, warnings and precautions. As part of its continuous product development policy, Cardinal Health reserves the right to change product specifications without prior notification. Please contact your Cardinal Health representative for additional product availability information. © 2018 Cardinal Health. All Rights Reserved. CARDINAL HEALTH, the Cardinal Health LOGO, ESSENTIAL TO CARE, FLEX ADVANTAGE and MEDI-VAC are trademarks of Cardinal Health and may be registered in the US and/or in other countries. Lit. No. 2MC18-867245 (11/2018)


CardinalHealth
Essential to care™