





# Medi-Vac Flex™ Advantage™ Tandem-Canister


## Set-up process


-  Before use, visually inspect the liner for perforation or other damage. Also, inspect the reusable outer canister for wear or other damage. Discard product if damage is observed.
-  Do not use if product is past its "Use by" date.


# Set-up


- 


Place Cardinal Health™ Medi-Vac™ Reusable Outer Canisters into bracket on roll stand or wall mount to be used in vertical orientation only.
- 


Remove liner from the packaging, then fully extend and twist the flexible liner. *(repeat with each liner)*
- 


Insert the liner into the same size Cardinal Health™ Reusable Outer Canister. *(repeat with each canister)*
- 


Press down firmly on the lid to ensure proper seal and listen for a "click". *(repeat with each canister)*
- 

Connect the red, braided tube to the VACUUM Port using a "push and twist" motion. *(repeat with each canister)*
- 

Attach the patient tubing to the PATIENT Port on first canister and remove the 90 degree elbows from all remaining liners.
- 

Starting with the first liner that has patient tubing attached and insert the male end of the tandem tube into the tandem port. Verify that the connector is inserted fully and locked into place. *(listen for a click)*
- 


Attach the female end of the tandem tube to the patient port on the next liner. Press down firmly to seal the connection.
- 


Repeat these steps until all Medi-Vac™ Flex Advantage liners are connected. Verify that the tandem port is securely capped on the final liner.
- 


Verify that all the connections are secure, and caps are sealed. Turn suction ON using on/off valve on the Medi-Vac™ Flex Advantage™ Bracket or the regulator valve.


# Medi-Vac Flex™ Flex Advantage™ Tandem-Canister


## Shutdown process


- 

With suction still ON, on the first liner, disconnect the patient tubing from the PATIENT Port and securely cap the PATIENT Port.
- 

With suction still ON, on the 2nd liner, disconnect the female end of the tandem tube and using the attached cap, ensure a secure seal. Remember to close the patient port on the Medi-Vac™ Flex Advantage™ liner. Repeat these steps for all the remaining liners.
- 

With suction still ON, disconnect the red, braided tube from the VACUUM Port and securely cap the VACUUM Port. Repeat these steps for all the remaining liners.
- 

Ensure vacuum remains ON until all ports are securely capped. Turn suction OFF using on/off valve on the Medi-Vac™ Flex Advantage™ Bracket or the regulator valve.
- 

Remove liner by lifting upward on the rim of the lid. Liners are disposable and should not be reused. Dispose of in compliance with hospital protocol.
- 

Do not discard the reusable outer canister. Reusable components should be cleaned and disinfected in accordance with facility procedures.

# Shutdown

For healthcare professionals only.

Important information: Prior to use, refer to the instruction for use supplied with this device for indications, contraindications, side effects, suggested procedure, warnings and precautions. As part of its continuous product development policy, Cardinal Health reserves the right to change product specifications without prior notification. Please contact your Cardinal Health representative for additional product availability information. © 2018 Cardinal Health. All Rights Reserved. CARDINAL HEALTH, the Cardinal Health LOGO, ESSENTIAL TO CARE, FLEX ADVANTAGE and MEDI-VAC are trademarks of Cardinal Health and may be registered in the US and/or in other countries. Lit. No. 2MC18-867249 (11/2018)



CardinalHealth

Essential to care™