


VACUETTE® Blood Culture Safety Blood Collection Sets

The Safety Blood Collection Set with pre-attached holder has been designed for use with Blood Culture bottles and **VACUETTE®** blood collection tubes in one procedure, thus saving you time and money on additional butterfly sets, holders and adapters. Sterile and pre-assembled for ease of use, it ensures compliance with both local blood culture collection protocols, and with the recommendations of the **NHS Saving Lives Document: Taking Blood Cultures – A summary of best practice.**

The Blood Culture Safety Blood Collection Set is quick, cost effective and ensures your patient only has to undergo one venepuncture for all procedures.

- Sterile
- Safe
- Cost effective

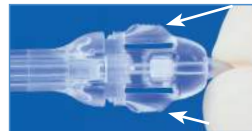

The safety protection mechanism is activated by pressing in both buttons of the safety hub and sliding backwards

Instructions for Use (ensure all components are securely fixed together)



1 Prepare venepuncture site and clean septum of Blood Culture Bottles in accordance with Trust protocol



2 Using the label scale as your guide, mark on the Blood Culture bottle the required fill level



3 Perform venepuncture



4 Push the holder down over the aerobic Blood Culture bottle, ensuring holder and bottle are below the puncture site. Once filled to required level, remove and repeat with anaerobic bottle. Continue with VACUETTE® Blood Collection Tubes (if required)




5 Activate the safety mechanism by pressing in both buttons of safety hub



6 Slide safety hub backwards until an audible click is heard



7 The needle will then be fully enclosed in the protective shield

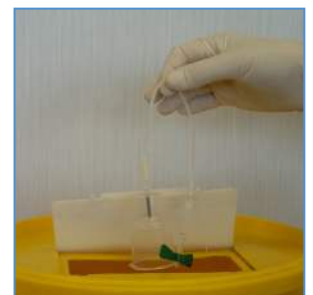

8 Dispose of in an approved sharps container

Ordering Details

Item No.	Description	Needle Gauge	Tube Length	Inner Pack	Outer Pack	NPC Code
450 182	Safety Blood Collection Set + Blood Culture Holder	21g	19cm	24	240	TBC
450 183	Safety Blood Collection Set + Blood Culture Holder	23g	19cm	24	240	TBC
450 184	Safety Blood Collection Set + Blood Culture Holder	21g	30cm	24	240	TBC
450 185	Safety Blood Collection Set + Blood Culture Holder	23g	30cm	24	240	TBC