

VACUETTE® Safety Blood Collection Sets - Instructions for Use

Activation before removal of needle from the vein




Select required safety blood collection set from **VACUETTE®** range and perform venepuncture in accordance with Trust guidelines.

Remove the Safety Blood Collection Set as shown below:



Ensure the wings of the butterfly set are secure with adhesive medical tape. Using one hand, cover the puncture site and the sliding shield with sterile gauze - with the other hand press in both sides of the safety hub.

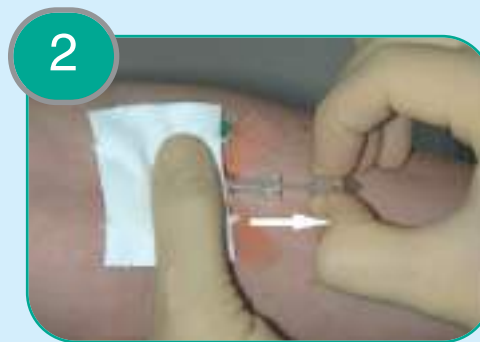

Slide the safety hub **backwards**.



An audible click confirms the shield is completely locked. The cannula should be fully enclosed in the protective shield.

Dispose of the used set in an appropriate sharps container.