

VACUETTE® Safety Options


Routine Venepuncture



Activate the safety shield with the aid of a solid surface.





An audible click can be heard, assuring the user that the safety shield has been properly activated

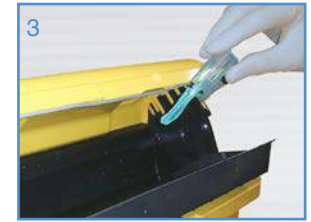

Dispose of the needle and the QUICKSHIELD holder in a sharps box

Difficult Veins

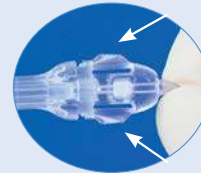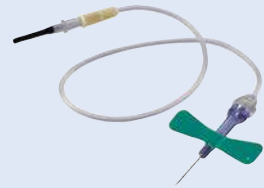

The safety protection mechanism is activated by pressing in both buttons of the safety hub and sliding backwards



Activate the safety mechanism by pressing in both buttons of safety hub



Slide safety hub backwards until an audible click is heard



The needle will be fully enclosed in the protective shield



Dispose of in an approved sharps container

In-situ Line and Cannulae



Holdex® Holder



For newly inserted Cannulae and In-Situ Lines, aspirate first

Transfer from a Syringe



Blood Transfer Unit



Safely remove and dispose of the needle from the syringe, firmly attach the BTU



Push the appropriate VACUETTE blood tube onto the needle



Allow tube to fill with the correct amount of blood then withdraw

NPC codes for VACUETTE® products

Product	Unit of Issue	NPC code
VISIO PLUS 'Flashback' Needle (21g)	100	KFK 023
VISIO PLUS 'Flashback' Needle (22g)	100	KFK 017
QUICKSHIELD Safety Holder	50	KFK 287
Standard Tube Holder	100	KFK 322
Safety Blood Collection Set with Luer Adapter (21g and 19cm tubing)	50	KFK 134
Safety Blood Collection Set with Luer Adapter (23g and 19cm tubing)	50	KFK 136
Safety Blood Collection Set with Luer Adapter (21g and 30cm tubing)	50	KFK 302
Holdex® Holder	100	KFK 111
Blood Transfer Unit	100	KFK 373