

Vastrip® 2+

The flexible stripper

Vastrip® 2+ is a product of excellent quality that can easily be adapted to different vein stripping techniques.

Vein stripping is a standard procedure in the treatment of varicose veins. It has a better long-term result than sclerotherapy (1) or saphenofemoral ligation alone (2). Vastrip 2+ is a popular and high quality vein stripper that has unique flexibility when vein surgery is being performed. Insertion can be performed in either direction using either the straight or spiral formed end.

If conventional stripping is required then the selected olive can be placed on either end of the stripper allowing pull through or pull back stripping.

Alternatively Vastrip 2+ can be used for invagination or mini-stripping by placing a ligature through the hole in the tip.

The handle can be positioned and repositioned on the Vastrip 2+ when stripping the vein. A sterile holder keeps the stripper and handle in place until time of use.

- **Suitable for different vein stripping techniques: conventional, invagination or mini-stripping**
- **Easy to introduce and control during the whole operation**
- **Two-way stripper with spiral and straight formed tips**
- **Three olives included (9, 12, 15 mm)**
- **An adjustable handle gives extra pulling strength**

Vastrip® 2+


Holder

Vastrip 2+ is supplied on a sterile holder.




Olives


There is a choice of 3 olive sizes.

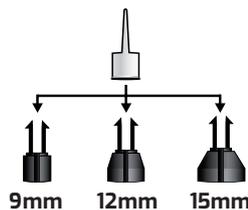

Straight tip

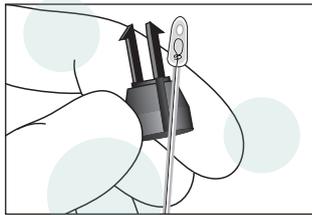



Spiral tip

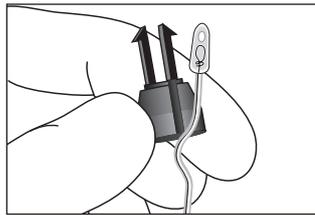

Tips are either straight or spiral and have a hole to facilitate the use of a ligature for invagination stripping.

Handle

One of the unique features of Vastrip 2+ is the movable handle. The handle gives extra pulling strength when vein stripping is performed.

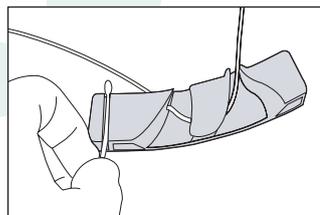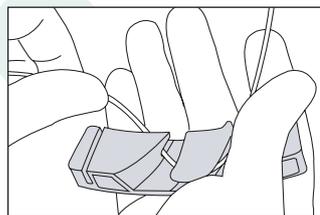

Description

Material

Stainless steel (stripper core), Polycarbonate (stripper sheath and tips), Acetale (olives), ABS (handle).

Sterilisation

The products are sterilised using ethylene oxide.

Single use

All components are intended for single use only. Do not use if the package is open or damaged.

Product range – Vastrip® 2+

Ref. nr 67321 4x10

References:

1. Einarsson E, Eklöf B, Neglén P. Sclerotherapy or surgery as treatment for varicose veins: a prospective randomized study. *Phlebology* 1993;8:22-26.
2. Dwerryhouse S, Stripping the long saphenous vein reduces the rate of reoperation for recurrent varicose veins: five-year results of a randomized trial, *Journal of Vascular Surgery*, 1999;29:589-92.