

Cutimed® Sorbact®

Closing wounds. Together.

**Need safe and
efficient infection
prevention and
management?**

The Cutimed®
Sorbact® range.
A responsible choice.



TOGETHER WE CAN MAKE THE CHANGE TO SAFE INFECTION PREVENTION AND MANAGEMENT

Having to **compromise between safety and efficiency is no longer** an issue for you and your patients. Commonly used silver-based dressings have their challenges, such as the risk of developing resistance or healing delays, **Cutimed® Sorbact®** performs **without any known side effects**.

IT'S TIME TO CHANGE.

IT'S TIME TO GO GREEN!

Due to the **purely physical mode of action**, **Cutimed® Sorbact®** binds and removes bacteria without the release of any antimicrobial agents. So experience the efficacy of **Cutimed® Sorbact®**.

PROVEN
clinical results

SAFE
No known
side effects or
contraindications

EASY
Shapes, sizes and
variants for most
wound types



**For me, there is
no other way of
safely managing
wound infection.**


Cutimed® Sorbact®
is my preferred
choice over silver-
based dressings.

**WELCOME TO
GREENERATION**

CUTIMED® SORBACT®'S EFFICACY OUTSTANDING PROVEN RESULTS

Cutimed® Sorbact®'s performance on infected wounds is clinically proven.² It reduces bacterial burden in critically colonised wounds up to **73%**, significantly out-performing silver-based dressings

Cutimed® Sorbact® is more effective than silver²

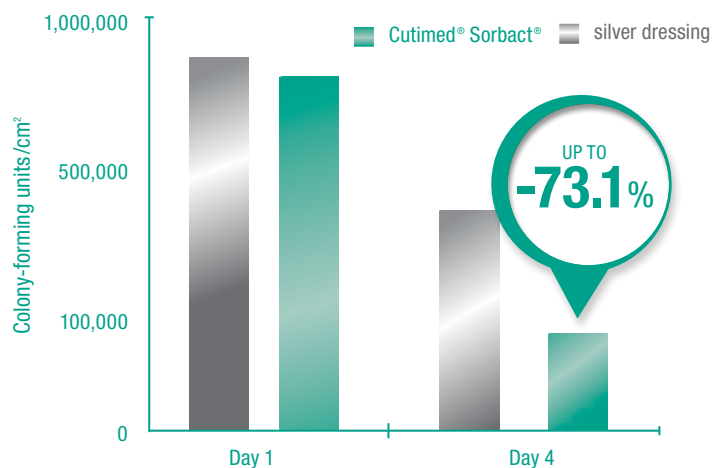

SILVER HAS MANY DRAWBACKS

Silver-based wound dressings can be harmful to your patient as they trigger the release of endotoxins and may have many side effects. **So make the switch to Cutimed® Sorbact® and avoid risks** that arise from using silver-based wound dressings:


- Killing microorganisms, e.g. by destroying cell walls³
- Healing impaired due to dead cell residue (e.g. endotoxins)⁴
- Possible side effects⁵
- Risk of resistant strains developing⁶
 - ▶ Not suitable for breastfeeding or pregnant women⁷




CUTIMED® SORBACT® SAFELY BINDS BACTERIA PURELY PHYSICALLY



An example of a colonised or infected wound, treated with **Cutimed® Sorbact®**.

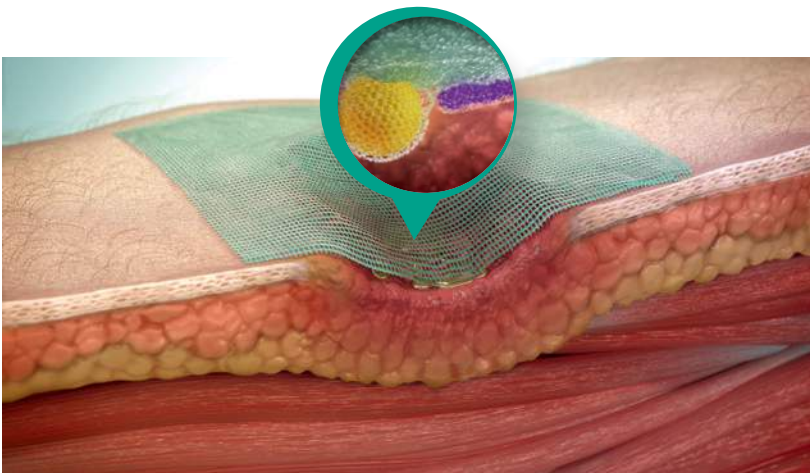

Cutimed® Sorbact® should be applied directly to the wound. Bacteria and / or fungi are attracted and bound to the **Sorbact®** mesh through hydrophobic interaction.

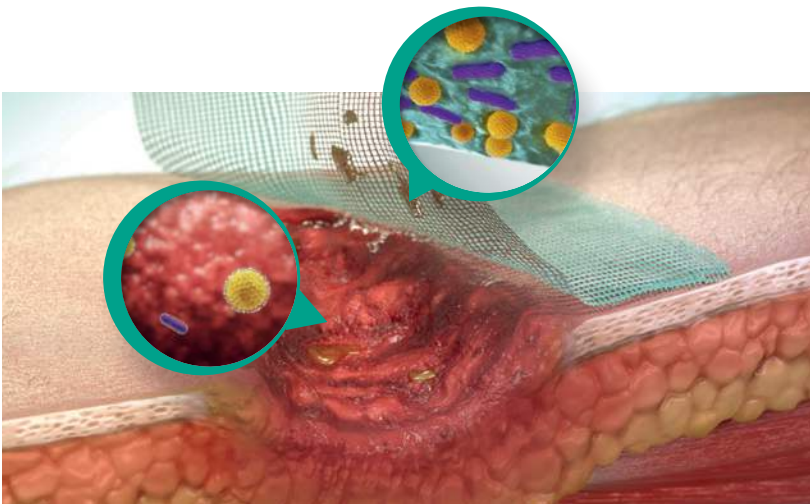

These bound pathogens are then removed with each dressing change, effectively reducing the microbial load.

To learn more about **Cutimed® Sorbact®**'s purely physical efficacy and mode of action, please visit our **Cutimed®** YouTube channel:





SAFE

NO SIDE EFFECTS KNOWN

Cutimed® Sorbact® has a purely physical mode of action.

This creates a variety of benefits, which make your **daily work much safer**:

- No contraindications known
- No known side effects
- No limitation in usage
- No release of antimicrobial agents
- Low risk of allergies

In addition, the dressings can be used for:

- Repeated usage
- Children
- Pregnant and breastfeeding women



EASY WITH CONVENIENT SOLUTIONS

1. Choose **Cutimed® Sorbact®** for **PREVENTION AND INFECTION MANAGEMENT**^{1,2}
2. Safe to use for **CHRONIC AND ACUTE WOUNDS** with no known contraindications
3. Select from different variants, shapes and sizes for easy **APPLICATION** across many wound types 
4. Treat **SUPERFICIAL AND DEEP WOUNDS** 

CUTIMED® SORBACT®
PROVIDES

ALL-ROUND
INFECTION MANAGEMENT



CUTIMED® SORBACT® HYDROACTIVE B

STIMULATES THE WOUND



Cutimed® Sorbact® Hydroactive B manages infection and stimulates **autolytic debridement** of fibrin and necrotic tissue.




Eases total treatment by combining autolytic debridement, exudate and infection management


Secure fixation
- hydrogel adhesive border

Supports autolytic debridement
via hydropolmer gel sheet

Manages and prevents infection thanks to the highly effective **Sorbact®** wound contact layer

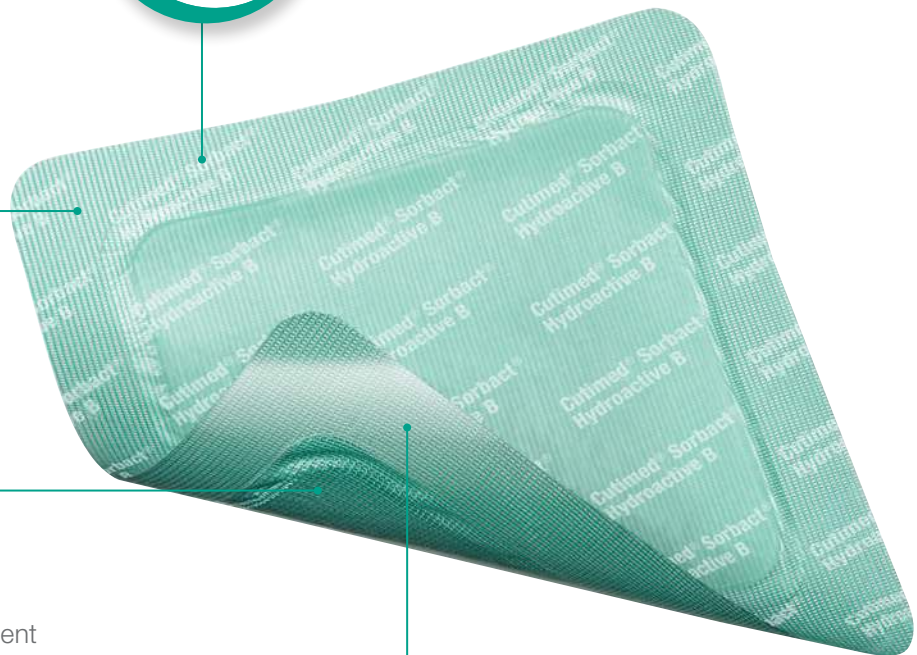

CUTIMED® SORBION® SORBACT® **POWERFUL** EXUDATE MANAGEMENT

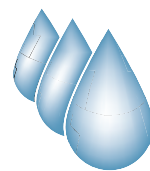

Cutimed® Sorbion® Sorbact® enables safe infection and exudate management, and is capable of handling a **large volume of fluid** thanks to the super-absorbent core.

Increased patient comfort and peace of mind thanks to anti-strike-through backing layer

Outstanding exudate management through super-absorbent core, even under pressure


50%
INCREASED ABSORPTION
CAPACITY*

Optimised fluid distribution thanks to non-woven distribution layer

Manages and prevents infection thanks to the highly effective **Sorbact®** wound contact layer

* In comparison to competitive super-absorbers – data on file.

CUTIMED® SILTEC® SORBACT® **BACTERIA** **BINDING** FOAM DRESSING

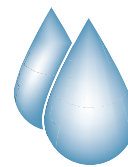


Cutimed® Siltec® Sorbact® combines the benefits of both infection and exudate management by using a **bacteria-binding wound contact layer** and an **innovative, gentle foam**. It is suitable to manage the bacterial load in moderate to highly exuding wounds.

NEW

Ideal healing conditions due to moisture-balancing, highly breathable polyurethane film

Protects wound edges with vertical absorption through polyurethane foam

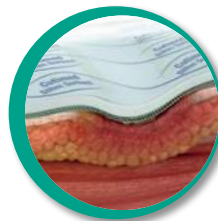

Increased flexibility of absorption capacity due to super-absorbent hot-melt stripes, for fluid retention

Manages and prevents infection thanks to the highly effective **Sorbact®** wound contact layer




Gentle fixation
silicone border

INDIVIDUAL AND EASY IN ITS APPLICATION

Cutimed® Siltec® Sorbact® hints and tips, for secure fixation on hard to dress areas.




To alter the shape of the dressing you can make additional cuts within the fixation border – not the foam.

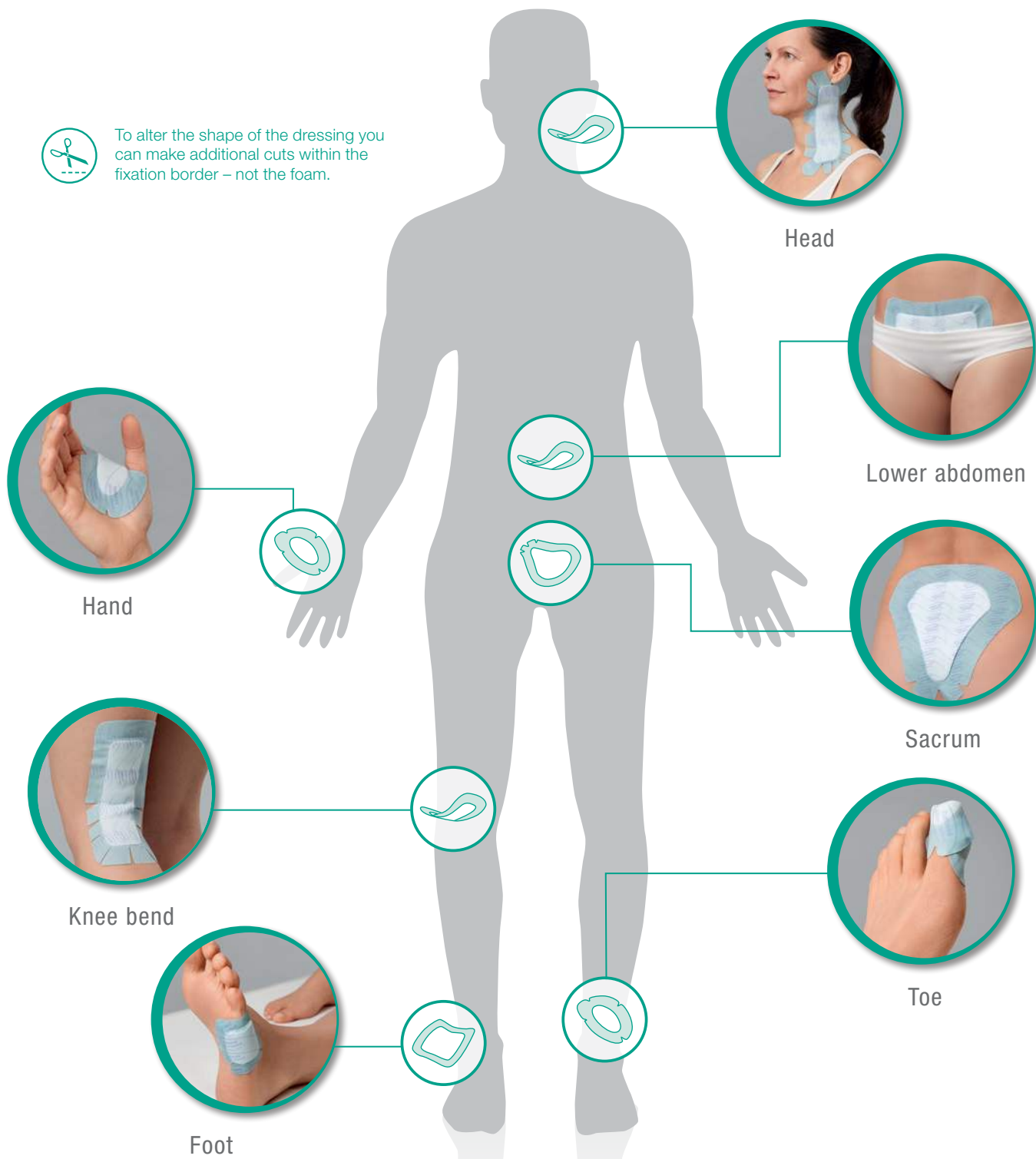

THE CUTIMED® SORBACT® RANGE

FULL RANGE SELECTION GUIDE

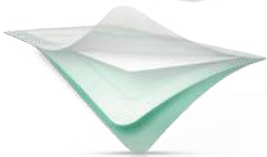Product		Description
Cutimed® Sorbact® Swabs 		An acetate swab coated with Sorbact® technology.
Cutimed® Sorbact® Dressing Pads 		A Sorbact® technology coated swab with an absorbent pad.
Cutimed® Sorbact® Ribbon 		Sorbact® technology coated cotton gauze.
Cutimed® Sorbact® Round Swabs 		Sorbact® technology coated acetate fabric.
Cutimed® Sorbact® Gel 		A Sorbact® technology coated swab coated with an amorphous hydrogel.
Cutimed® Sorbact® Hydroactive Cutimed® Sorbact® Hydroactive B 		A Sorbact® technology coated swab combined with an absorbent, hydropolymer gel sheet.
Cutimed® Siltec® Sorbact® 		A Sorbact® technology coated swab combined with a Cutimed® Siltec® B foam dressing.
Cutimed® Sorbion® Sorbact® 		A Sorbact® technology coated super-absorbent dressing with strike through prevention backing layer.
Leukomed® Sorbact®		A Sorbact® technology coated post operative dressing with a waterproof film.

*When using a secondary dressing, this may need to be changed more frequently. For infected wounds, it is important to monitor the wound daily.

When to use	Method of use	Frequency of change
Primary dressing for contaminated, colonised or infected superficial or deep wounds. Also suitable for fungal infection in the groin, in skin folds (e.g. under the breast) or between digits.	Primary dressing for either deep or superficial wounds. Swab dressings are sold as folded but are primarily used unfolded as an effective antimicrobial wound contact layer. Secondary dressing required.	Can be left in place for up to seven days. Please use clinical judgement depending on the wound condition.*
Suitable for all types of contaminated, colonised or infected wounds with low to moderate exudate levels.	Apply the green side of the dressing to the wound and retain with a bandage, tape or film dressing. Cannot be cut.	Can be left in place for up to seven days. Please use clinical judgement depending on the wound condition.*
All types of deep or cavity wounds showing signs of infection. Also suitable for fungal infection in the groin, in skin folds (e.g. under the breast) or between the digits.	Fill the wound with the dressing or place around digits or in skin folds if fungal infections are a problem.	Can be left in place for up to seven days. Please use clinical judgement depending on the wound condition.*
Use for dressing small, deep wounds.	Retain with a secondary dressing.	Can be left in place for up to seven days. Please use clinical judgement depending on the wound condition.*
Can be used for all types of dry, sloughy or low exuding wounds that are contaminated, colonised or infected.	Apply directly to the wound. Cover with an appropriate secondary dressing that does not absorb the hydrogel.	Recommended dressing change interval is one - three days depending on the wound exudate levels.*
Can be used for low to moderately exuding wounds that are contaminated, colonised or infected. Also supports autolytic debridement and helps to maintain a moist wound environment.	Use a bandage, tape or film to secure Cutimed® Sorbact® Hydroactive dressings. Cutimed® Sorbact® Hydroactive B has an atraumatic adhesive border - no additional fixation required.	The dressings should be changed as often as dictated by the wound condition or between one and four days. Change when the dressing is saturated (becomes cloudy or opaque).*
For moderate to highly exuding, contaminated, colonised or infected wounds. Can also be used as a secondary dressing for infected deep or cavity wounds.	Apply the pad to the wound. The dressings have a silicone adhesive border for retention.	The dressing should be changed as often as dictated by the wound condition or when the dressing is saturated. Usually between one and three days.*
For infected wounds or those at risk of infection with high to very high levels of exudate.	Apply ensuring green side of dressing has direct contact with wound bed, use appropriate fixation.	Dependent on condition of wound and clinical judgement. If conditions allow, may stay in place of up to seven days.
For surgical incisions, post operative dehisced wounds, lacerations, cuts and abrasions with low to moderate exudate levels.	Apply with green pad in direct contact with wound, smooth film securely to the skin and carefully remove application film using the blue handling strip.	Dependent on wound condition i.e. exudate level and presence of infection. Dressing should be changed at least twice a week.



**You don't have to
choose a dressing
with side effects
any more**



Only
Cutimed® Sorbact®
is free from
side effects.*

References

- ¹ Stanirowski et al., (2016) "Randomized controlled trial evaluating dialkylcarbamoyl chloride impregnated dressings for the prevention of surgical site infections in adult women undergoing caesarean section" Archives of Medical Science, 12(5): 1036-1042
- ² Mosti et al., (2015) "Comparative study of two antimicrobial dressings in infected leg ulcers: a pilot study", Journal of Wound Care, 24(3): 121-2; 147-7
- ³ Hermans MH. (2007) Silver-containing dressings and the need for evidence. Advances in Skin & Wound Care, Vol 20, Issue 3
- ⁴ Ovington L. (2003) Bacterial toxins and wound healing. Ostomy Wound Management 49(7A Suppl): 8-12
- ⁵ Lansdown AB, Williams A. (2004) How safe is silver in wound care? Journal of Wound Care, 13(4):131-6
- ⁶ Finley, Durnham, (2014) Evidence of emergent silver-resistance in clinical bacteria: A major implication of wound care and the use of silver dressings, Presented at SAWC Fall Las Vegas
- ⁷ Wounds UK (2013) Best Practice Statement. The use of topical antimicrobial agents in wound management. London: Wounds UK (third edition).
- ⁸ Burd, A., Kwok, C. H., Hung, S. C., Chan, H. S., Gu, H., Lam, W. K. and Huang, L. (2007), A comparative study of the cytotoxicity of silver-based dressings in monolayer cell, tissue explant, and animal models. Wound Repair and Regeneration, 15: 94-104.

*Up to now there are no known side effects.

Ordering information

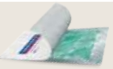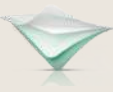Cutimed® Sorbact®	Wound pad	Size	BSN Code	Dressings per box	NHS SC Code	PIP Code
Dressing pads Dressing fabric with absorbent core 		7 x 9 cm	72161-01	1 x 5	ELY214	232-4291
		7 x 9 cm	72161-00	1 x 40	-	-
		10 x 10 cm	72162-01	1 x 5	ELY215	232-4309
		10 x 10 cm	72162-00	1 x 40	-	-
		10 x 20 cm	72163-00	1 x 20	ELY219	232-4317
Ribbon gauzes For use in cavity wounds of all kinds 		2 x 50 cm	72166-00	1 x 20	ELY218	339-9482
		5 x 200 cm	72167-00	1 x 10	ELY217	339-9490
Swabs Flat, folded dressing for deep or superficial wounds 		4 x 6 cm	72164-01	1 x 5	ELY212	232-4325
		4 x 6 cm	72164-00	1 x 40		
		7 x 9 cm	72165-01	1 x 5	ELY213	232-4333
		7 x 9 cm	72165-00	1 x 40		
Round Swabs Preformed for deep and superficial wounds 		3 cm	72168-00	1 x 70 (14 x 5)	ELY309	232-4341
Sorbact® Gel Promotes autolytic debridement 		7.5 x 7.5 cm	72611-00	1 x 10	ELY338	347-7015
		7.5 x 15 cm	72611-01	1 x 10	ELY339	347-7023
Sorbact® Hydroactive Absorbent hydrogel core for low to moderately exuding wounds 	(5 x 6.5 cm)	7 x 8.5 cm	72646-03	1 x 10	ELY347	362-9961
	(10 x 10 cm)	14 x 14 cm	72646-04	1 x 10	ELY348	362-9979
	(15 x 15 cm)	19 x 19 cm	72646-05	1 x 10	ELY349	362-9987
	(10 x 20 cm)	14 x 24 cm	72646-07	1 x 10	ELY351	363-0001
	(20 x 20 cm)	24 x 24 cm	72646-06	1 x 5	ELY350	362-9995
Sorbact® Hydroactive B (border) Absorbent hydrogel core for low to moderately exuding wounds; adhesive border 	(5 x 6.5 cm)	7 x 8.5 cm	79933-00	1 x 10	ELY500	374-5023
	(10 x 10 cm)	14 x 14 cm	79933-02	1 x 10	ELY501	374-5031
	(15 x 15 cm)	19 x 19 cm	79933-03	1 x 10	ELY502	374-5049
	(10 x 20 cm)	14 x 24 cm	79933-04	1 x 10	ELY503	374-5056
Siltec® Sorbact® Super-absorbent foam dressing for low to moderately exuding wounds; silicone adhesive border 	(4 x 4 cm)	7.5 x 7.5 cm	73251-00	1 x 10	TBC	377-3470
	(6 x 6 cm)	10 x 10 cm	73251-16	1 x 10	TBC	377-3488
	(5 x 16.5 cm)	10 x 22.5 cm	73251-14	1 x 10	TBC	377-3504
	(8.2 x 8.2 cm)	12.5 x 12.5 cm	73251-01	1 x 10	TBC	377-3520
	(10 x 10 cm)	15 x 15 cm	73251-02	1 x 10	TBC	377-3538
	(12.3 x 12.3 cm)	17.5 x 17.5 cm	73251-03	1 x 5	TBC	377-3546
	(16.5 x 16.5 cm)	22.5 x 22.5 cm	73251-04	1 x 5	TBC	377-3553
Siltec® Sorbact® (oval)	(3 x 6 cm)	7 x 10 cm	73251-18	1 x 10	TBC	TBC
Siltec® Sorbact® (sacrum)		17.5 x 17.5 cm	73251-05	1 x 5	TBC	TBC
		23 x 23 cm	73251-06	1 x 5	TBC	TBC
Sorbion® Sorbact® Super-absorbent dressing for highly exuding wounds 	(8 x 9 cm)	10 x 10 cm	72698-00	1 x 10	TBC	TBC
	(7.7 x 17.8 cm)	10 x 20 cm	72698-01	1 x 10	TBC	TBC
	(17.8 x 17.8 cm)	20 x 20 cm	72698-02	1 x 10	TBC	TBC
	(17.5 x 27.5 cm)	20 x 30 cm	72698-03	1 x 10	TBC	TBC
Leukomed® Sorbact®						
Especially suitable for use in post-operative wounds 	(2.8 x 3.8 cm)	5 x 7 cm	76199-00	1 x 20	ELY578	TBC
	(4 x 6.5 cm)	8 x 10 cm	76199-01	1 x 20	ELY579	TBC
	(4 x 11 cm)	8 x 15 cm	76199-02	1 x 20	ELY580	TBC
	(5 x 15 cm)	10 x 20 cm	76199-03	1 x 20	ELY581	TBC
	(5 x 20 cm)	10 x 25 cm	76199-04	1 x 20	ELY582	TBC
	(5 x 25 cm)	10 x 30 cm	76199-05	1 x 20	ELY583	TBC
	(5 x 30 cm)	10 x 35 cm	76199-06	1 x 20	ELY584	TBC
	(2.8 x 3.8 cm)	5 x 7.2 cm	76199-07	1 x 3	ELY585	TBC
	(4 x 6.5 cm)	8 x 10 cm	76199-08	1 x 3	ELY586	TBC

Sorbact® is a registered trademark of ABIGO Medical AB.

Contact your local Account Manager or email concierge.service@essity.com to request an evaluation sample. **It's time to go green!**




Infection management

by Cutimed®

Cutimed® Sorbact® is suitable for clean, colonised and infected wounds such as:

- Chronic wounds (e.g. venous, arterial, diabetic or pressure ulcers)
- Postoperative dehiscent wounds
- Traumatic wounds
- Wounds post excision of abscesses
- Dermal fungal infections

EXUDATE
LEVEL

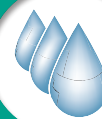

Infected wounds



superficial
wounds



deep
wounds



superficial and
deep wounds

WOUND BED PREPARATION




Cutimed® Sorbact® Hydroactive B



Cutimed® Siltec® Sorbact®



Cutimed® Sorbact® dressing pads

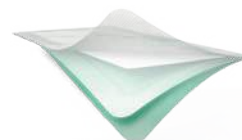

Cutimed® Sorbion® Sorbact®



Cutimed® Sorbact® ribbon gauzes



Cutimed® Sorbact® (round) swabs



Cutimed® Sorbact® gel*

*Promotes autolytic debridement in sloughy or partially **necrotic wounds**.